

01/03/2021

## NEMS-A ABPM v2.7.2 Release Notes

- **Prepare NBP1, Oscar2 Holter changes:**
  - Max **Day, Night** BP Systolic/Diastolic – now smaller step **5** (instead of 10)
  - **Awake/Asleep** Time **Hour** - now smaller step **10m** (instead of 30m)
  - Max **Day, Night** BP Systolic/Diastolic values are now **Global** not patient-specific, so every time the values are changing it will always remember the last values.

	Systolic		Diastolic
Max BP day limits	140	/	90
Max BP night limits	120	/	80

- **Auto-Complete Patient-ID bug Fix.**
- **Save ABPM PDF Report** – using a quick save button, now you can save ABPM PDF report in your favorite location.



- **New Summary report** with Maximum and Minimum measurements, a summary of average values, and overall stats.
- **Add/Remove measurement by click row number**

49	30/05/2007 11:00:00	144	113	98	81	A
	05/11/2019 10:28:07	–	–	–	–	U 107

**Double Click to Hide/Show**